

# Zeus V2

## Compatibility Guide

For Qualified Personnel  
Version 4.0 / 2026.01.13



# Table of Contents

Zeus V2 hands	2
Control Systems	3
Elbow Systems	4
Powered Wrist Rotator	5
Power Systems	6
Lamination Ring	7

# Zeus V2 hands

This compatibility guide applies to all Zeus V2 hands listed below:

- [A02L-SF0B] Zeus V2 Bionic Hand Small Left
- [A02R-SF0B] Zeus V2 Bionic Hand Small Right
- [A02L-MF0B] Zeus V2 Bionic Hand Medium Left
- [A02R-MF0B] Zeus V2 Bionic Hand Medium Right
- [A02L-SF1B] Zeus V2 Bionic Hand Small Flex Left
- [A02R-SF1B] Zeus V2 Bionic Hand Small Flex Right
- [A02L-MF1B] Zeus V2 Bionic Hand Medium Flex Left
- [A02R-MF1B] Zeus V2 Bionic Hand Medium Flex Right

# Control Systems

## General electrodes

Zeus V2 hands are compatible with all industry standard EMG electrodes with a signal output range of 0-5V, supports power input up to 8.4V, and features a 3-pin Kidney Connector interface.

## Pattern Recognition System

Zeus V2 hands are compatible with the pattern recognition systems noted below.

Manufacturer	Model
Coapt Engineering	Complete Control System Gen2
Infinite Biomed Technologies (IBT)	IBT Glide

# Elbow Systems

Zeus V2 hands are compatible with several elbow devices. Compatible elbow systems include those listed below.

Manufacturer	Model
Fillauer	Motion E2 Elbow
Ottobock	ErgoArm family
Ottobock	DynamicArm family

# Powered Wrist Rotator

Zeus V2 hands are compatible with the wrist systems listed below.

<b>Manufacturer</b>	<b>Model</b>
Fillauer	MC ProWrist Rotator
Fillauer	MC Standard Wrist Rotator
Ottobock	Electric wrist rotator
Ottobock	Electric wrist rotator + MyoRotronic

# Power Systems

Zeus V2 hands are compatible with 3rd party power systems that meet the following specifications:

1. Provide nominal 7.4 Volts (Li-Po/Li-Ion)
2. Are certified to the appropriate safety standards (IEC 62133 or IEC 60086-4) and UN38.3
3. Can supply a peak discharge current of at least 6.5A

<b>Manufacturer</b>	<b>Model</b>
Aether Biomedical	Aether Battery System
Infinite Biomed Technologies (IBT)	IBT FlexCell *(with at least 2 cells connected)
Infinite Biomed Technologies (IBT)	IBT FlexCell Mini *(with at least 2 cells connected)
Ottobock	Ottobock MyoEnergy Integral
Össur	2000 mAh Battery
Össur	1300 mAh Battery

To ensure a full day of usage without charging we recommend using batteries with capacity of at least 2000mAh

# Lamination Ring

Zeus V2 hands are compatible with the lamination rings listed below.

<b>Manufacturer</b>	<b>Model</b>
Steeper	Steeper Lamination Ring
Fillauer	Fillauer Lamination Ring
Ottobock	Ottobock Lamination Ring

**AETHER**  
B I O M E D I C A L  
[www.aetherbiomedical.com](http://www.aetherbiomedical.com)



Aether Biomedical Sp. z o.o.  
ul. Mostowa 11, 61-854  
Poznań

POLAND

2-B 75, Lajpat Nagar 2,  
New Delhi-110024  
INDIA

874 Walker Road, Suite C  
Dover, Delaware 19904  
Aether US Inc

phone:

POLAND +48 780011548  
INDIA +91 9650488846  
USA +1 4708238221

e-mail: [support@aetherbiomedical.com](mailto:support@aetherbiomedical.com)

©Copyright Aether Biomedical 2025. All right reserved.